

ANGIO Mentor Simulator Platforms

COVID-19 Disinfection Protocol

When using the simulator, it is recommended to use protection equipment (disposable gloves and face mask).

Prior to Cleaning Simulator:

1. Power down the simulator and disconnect AC power.
2. Put on disposable gloves.

Cleaning Simulator Parts:

1. Dampen a new clean microfiber cloth with the recommended disinfectant (see table below). Ensure excess liquid is squeezed from the cloth. You may also use pre-dampened isopropyl alcohol wipes.
2. Gently wipe the moistened surfaces to be cleaned.

Warning:

- Never clean a simulator while it is powered on or plugged in.
- Never spray/spill liquids of any kind directly onto the simulator.

ANGIO Mentor Flex II




ANGIO Tab Pro




Simulator Part	Disinfectant
<ul style="list-style-type: none"> 1 Touchscreen 3 UI Monitor 	Standard IPA (Isopropyl Alcohol) at 70% alcohol concentration for cleaning and disinfecting. DO NOT use IPA at concentration >70%
<ul style="list-style-type: none"> 2 Flex II Box, Joysticks and Buttons 4 Box-Leg cable 5 Simulation Units 6 Peripheral Devices 7 Keyboard 	Standard IPA (Isopropyl Alcohol) for cleaning and disinfecting (at least 70%)

ANGIO Mentor Suite II

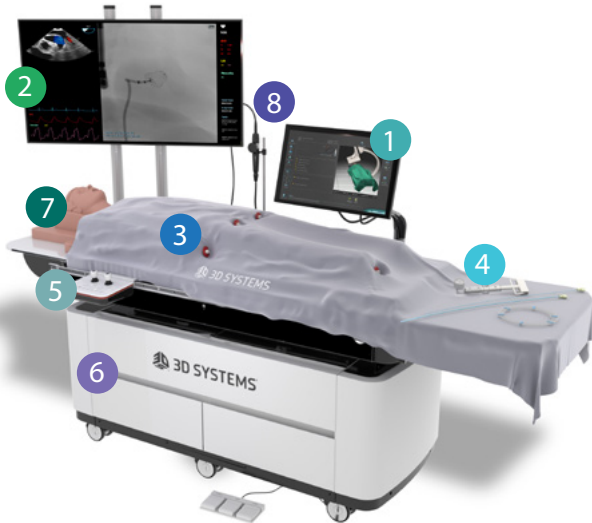

ANGIO Mentor Suite

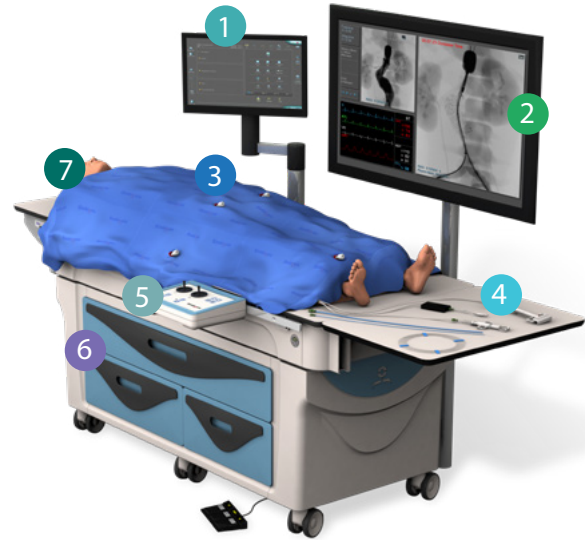

Simulator Part	Disinfectant
<ul style="list-style-type: none"> 1 Touchscreen 2 Fluoroscopy Screen LCD 	Standard IPA (Isopropyl Alcohol) at 70% alcohol concentration for cleaning and disinfecting. DO NOT use IPA at concentration >70%
<ul style="list-style-type: none"> 3 Life-like surgical drape with insertion holes 	Cover with disposable sterile surgical drape or a PVC shrink wraps. Create holes in the access sites locations.
<ul style="list-style-type: none"> 4 Peripheral tools and deployment handles 5 Control box 6 Drawers 7 Mannequin 8 TEE hanger and probe 	Standard IPA (Isopropyl Alcohol) for cleaning and disinfecting (at least 70%)

After Cleaning:

1. Make sure there is no visible moisture on the surfaces of the simulator before it is powered on or plugged in.
2. Remove and dispose gloves.
3. Wash your hands immediately after gloves are disposed.

Disclaimer: Though this disinfection protocol follows the guidance published by the [CDC](https://www.cdc.gov), 3D Systems cannot guarantee with 100% certainty that treatment per the above instructions will result in the total eradication of coronavirus.